


TUB & TRAY COVERS

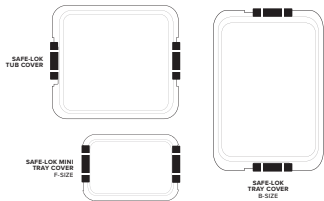

This product is made with polycarbonate.

CLEANING & DISINFECTION RECOMMENDATIONS

- Safe for use in an instrument washer disinfector
- Or may surface disinfect
- Read chemical labels carefully. Ensure chemical compatibility with polycarbonate products.
- Use pH-neutral* detergents/disinfectants or those with a pH <8.
 - This will help support longer life of the covers.
 - Alkaline detergents are NOT recommended.
- Rinse covers after disinfecting to help minimize degradation and cracking.

**Some pH-neutral detergents/disinfectants may still react with polycarbonate materials depending on chemical makeup.*



Made in the U.S.A.

Zirc Dental Products, Inc.
3918 Hwy 55 SE, Buffalo, MN, USA 55313
+1 (763) 682-6636 • www.zirc.com



Losser & Co. GmbH Benzstr. 1C, 51381 Leverkusen, Germany +49-2171-706670



20Z445IFU REV A 0223