


Multi-Mod Rack

***All of the below information is the same for both 6 and 8-Place Racks.*



Assembly:

Parts list:

- ① End Panel with handles (2 of each)
- ② Side Panel (2 of each)
- ③ Black Fasteners (4 of each)

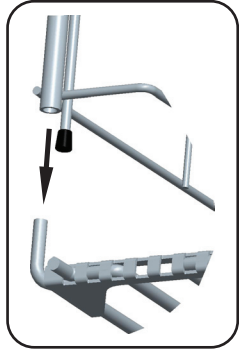


Slide inside of frame.

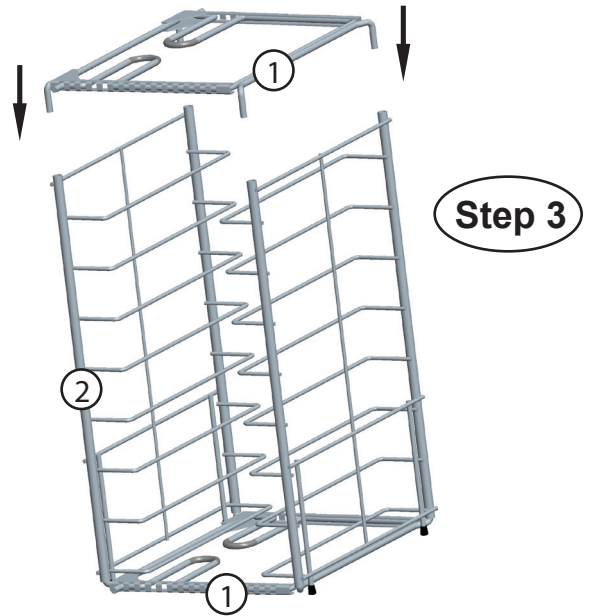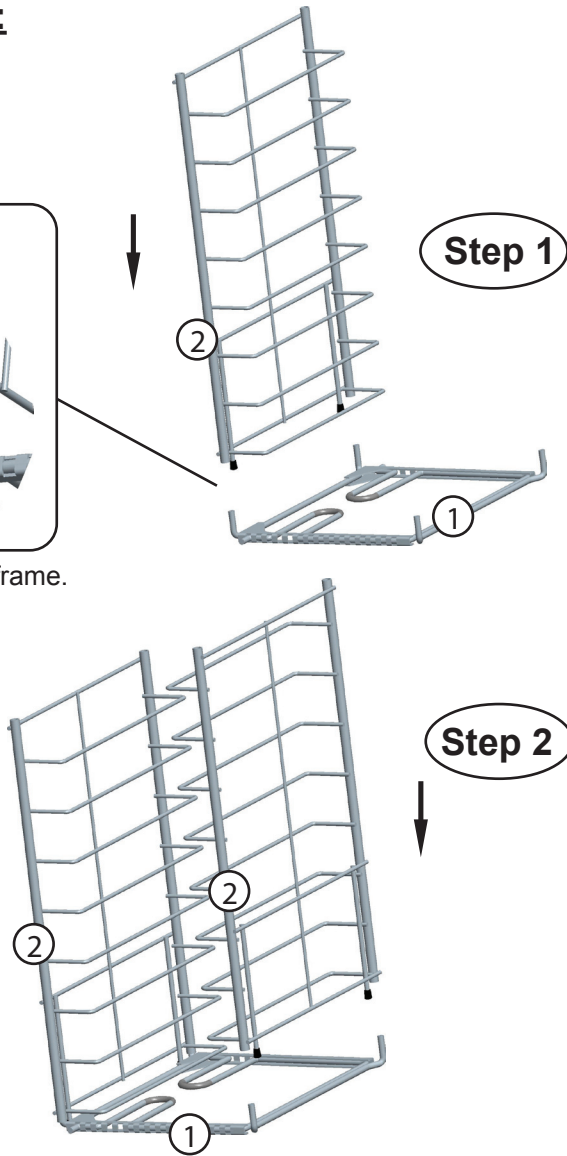



Assembly of Black Fasteners:

Attach black fasteners to connect the side panels to the end panels. Repeat steps 4 and 5 for both sides of the rack.

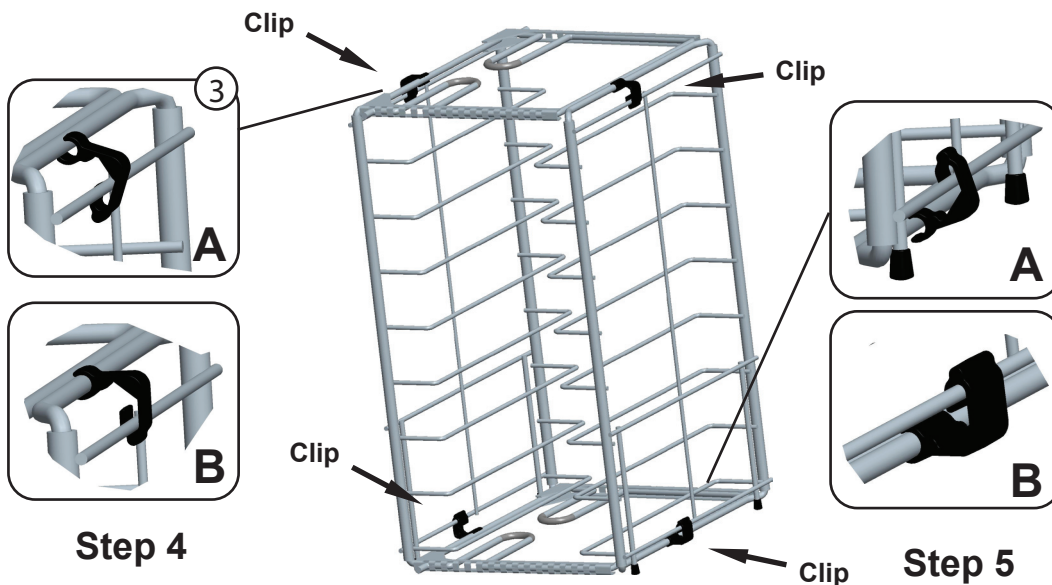

Adjusting Racks:

Lay the rack on its side with the handles facing you. Steady the rack in one hand and adjust with the other.

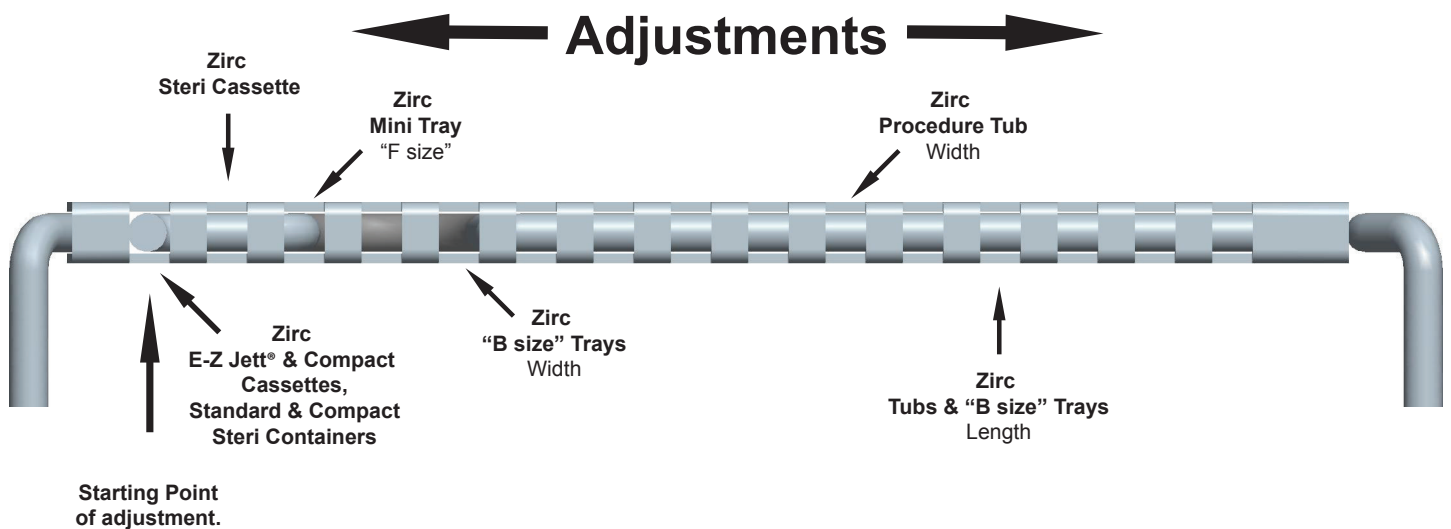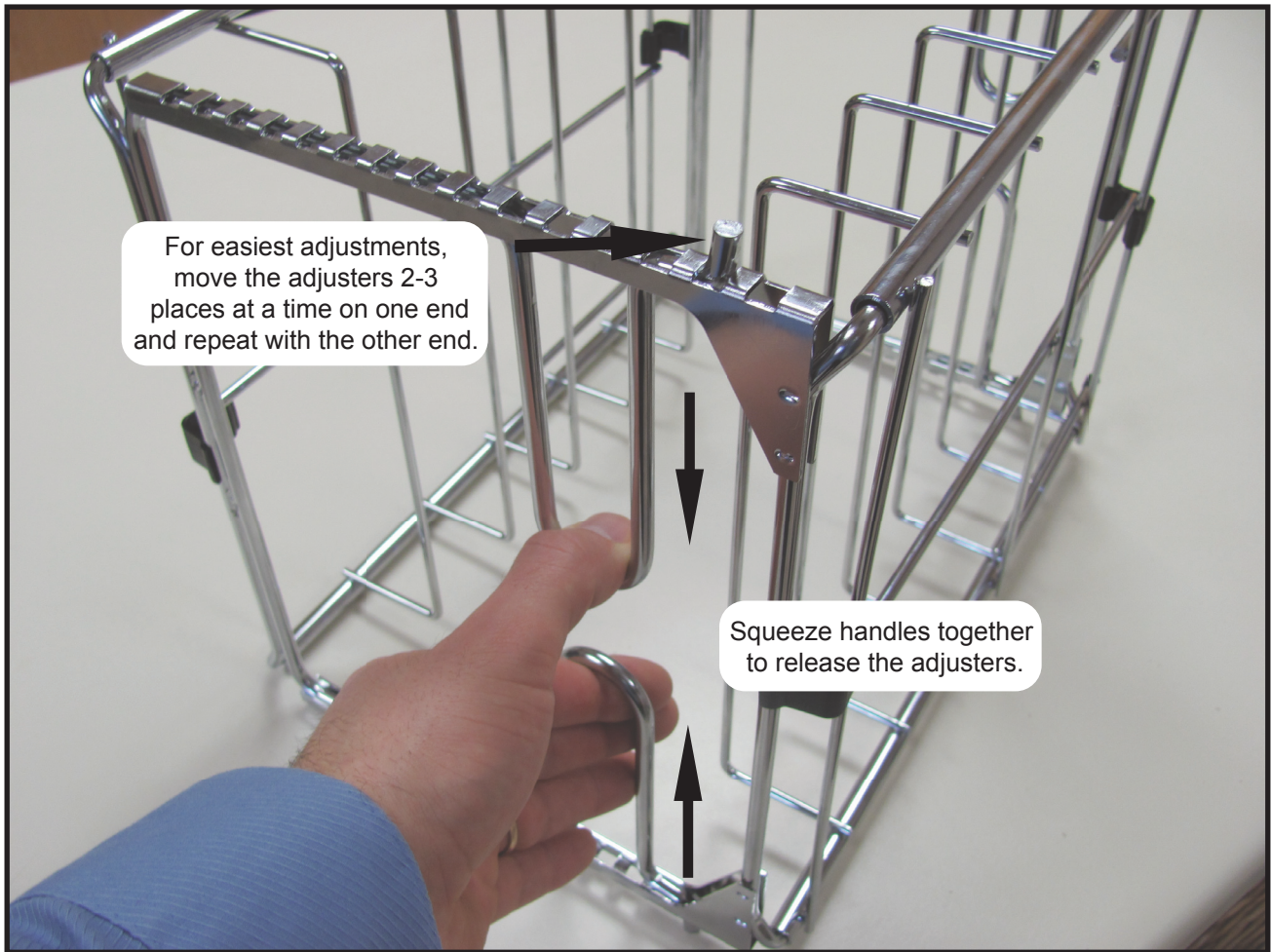

Have questions? Contact Zirc today!

800.328.3899 | 763.682.6636 | www.zirc.com


Zirc
Because Time Is Everything


 Loser & Co. GmbH Benzstr. 1-3,
D51381 Leverkusen Germany +49-2171-706670

21Z410LT REV B 02/12