


Water Wise

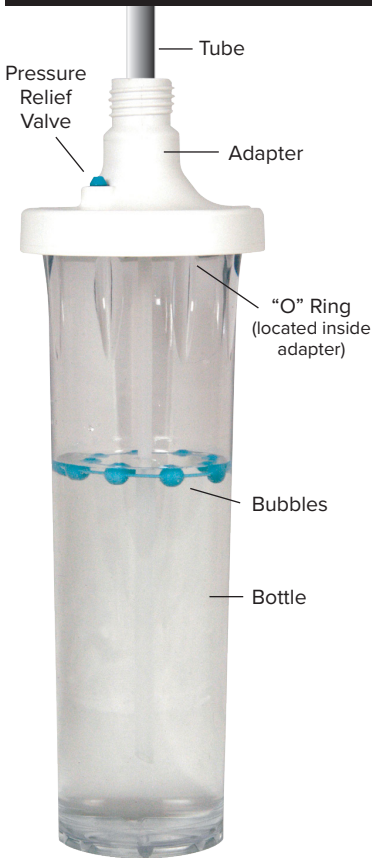

Instructions:

1. Remove the existing bottle and tube.
2. Install new tube on existing fitting.
3. Twist adapter securely into dental unit (make sure adapter is on tight).
4. Fill bottle.
5. Insert bottle into the adapter and turn right until locked.
6. To refill – only remove bottle. (Turn left to remove.)
7. **REPLACE BOTTLE ANNUALLY.**


BUBBLES (Water Level Indicator)

- Place bubble(s) in bottle for color-coding water type – distilled, medicament waters, etc.

LID

- Use to cover bottle during transportation and storage.
- Snap onto bottom to prevent misplacement.



 Manufacturer
Zirc Dental Products, Inc.
Buffalo, MN 55313
USA

Local: +1 763.682.6636

Fax: +1 763.682.6604

www.zirc.com


Zirc

Because Time Is Everything

23Z105IFU REV A 01/22

CAUTION:

MAXIMUM pressure shall not exceed **40lbs.**

Please Note: There is a blue pressure relief valve located on the adapter. If air pressure exceeds 45 lbs., the relief valve will raise to release pressure, and may result in hissing sounds.

If this occurs, we recommend contacting your service technician.