



E-Z Storage System

E-Z Storage Tub w/ White Cover	29R605
E-Z Storage Tub w/ Clear Cover	29R610
Single Rack (For E-Z Storage Tub)	29R650
Single Mounting Bracket for Rack	29R651

Accessories:

Full Divider	29R625
Half Divider	29R626
Hinges – (Pair)	29R627
Clear Cover Only	29R628
White Cover only	29R624



EC REP Loser & Co. GmbH Benzstr. 1c, D51381 Leverkusen Germany
+49-2171-706670

763-682-6636 • 800-328-3899 • Fax 763-682-6604
zirc@zirc.com • www.zirc.com

29R600LT Rev F 08/08



E-Z Access Storage System

Directions for Mounting Bracket (29R651)

To mount on Wall:

1. Locate a stud in wall
2. Position bracket against wall, center on stud
3. Mark 3 holes on wall with pencil
4. Drill outside holes with 3/16" drill bit
5. Drill center hole with 3/32" drill bit
6. Insert wall anchors into outer holes into wall anchors
7. Screw 2 short screws through outer holes into wall anchors
8. Screw long screw through center hole into wall stud

To Mount on Cabinet:

1. Position bracket on cabinet
2. Mark 2 outside holes with pencil
3. Drill outside holes with 3/32" drill bit
4. Screw 2 small screws through outside holes into cabinet

To use single rack with bracket, remove rubber feet from longest legs on rack; insert into bracket..