

# CAD/CAM BLOCK ORGANIZER




## FEATURES

- Fits all CAD/CAM milling blocks including PrograMill®, CEREC®, PlanMill™ blocks
- Provides a professional platform to present and discuss same-day restoration

## DIMENSIONS

5-9/16" x 1-7/8" x 7-3/4"

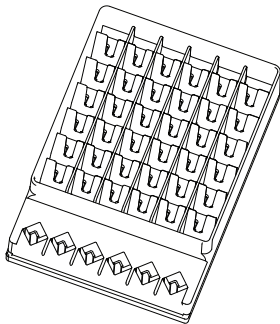

Made in the U.S.A.

Zirc Dental Products, Inc.  
3918 Hwy 55 SE, Buffalo, MN, USA 55313  
+1 (763) 682-6636 • [www.zirc.com](http://www.zirc.com)




EC REP Loser & Co. GmbH Benzstr. 1C, 51381 Leverkusen, Germany +49-2171-706670



50Z417IFU REV A 0323

# CAD/CAM BLOCK ORGANIZER

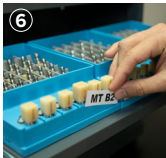

## HOW TO SET UP

- 1 - Individual/block storage: use gray block divider.
- 2 - Place individual milling blocks inside block divider by placing block side down.
- 3 - Block package storage: use gray slide by inserting feet into open channels.
- 4 - Push base against packages at the base to lock in place.
- 5 - Use front row to place individual blocks.
- 6 - Use the flat front of the product to label. Scan for label template. →

