E-Z Jett® Cassettes

Product Demo Videos available online:
<https://www.youtube.com/user/ZircDental>

Zirc Company

3918 Hwy 55 S.E. | Buffalo, MN 55313 | USA
Toll Free: 800.328.3899 | Local: 763.682.6636
www.zirc.com


Features:

- Made of durable, autoclavable, resin material
- Contains antimicrobial protection

****Do not allow cassette to touch side walls or top/bottom of sterilizer***

Squeeze tabs and lift to release cover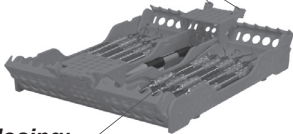

Closing:
Pegs on cover should be in grooves on cassette


10-Place
(50Z930)

Squeeze tabs and lift to release cover


8-Place
(50Z927)

Squeeze tabs and lift to release cover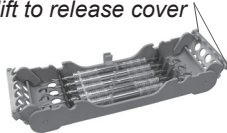

5-Place
(50Z925)

Pull up cover to release

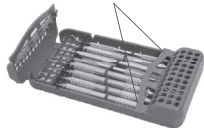8-Place Slim
(50Z911)

Zirc Company

Processing of Resterilizable Medical Devices

Manufacturer/ Hersteller: Zirc Company, 3918 Hwy 55 SE, Buffalo, MN 55313 USA

- 1. Restriction of Reprocessing:** Frequent reprocessing has marginal consequences for Zirc products. Normally, the end of durability of the product will be determined by visual degradation.
- 2. Cleaning:** An Ultrasonic, Instrument Washer or Thermal Disinfectant may be used. Follow manufacturer's directions for proper cleaning and use a mild alkaline detergent only. Rinse Zirc products with demineralized water to remove detergent or particles.
- 3. Packaging:** Bag or wrap product using sterilization pouch or sterilization wrap.
- 4. Sterilization:** Steam sterilize for at least 5 minutes at 273° F/ 134° C or 20 minutes at 250° F/ 121° C. Do not heat above 277° F/ 136° C. DO NOT dry heat sterilize.

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Losser & Co. GmbH Benzstr. 1-3, D51381 Leverkusen Germany +49-2171-706670